APHTO FIX

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AphtoFix mouth ulcer is a film-forming cream used for the local treatment of mouth ulcers, denture irritations and general gum irritations.

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EN Instructions for use

1. Wash your hands thoroughly before use.
2. Before application make sure that your hands are dry to avoid bringing moisture in contact with the tube nozzle.
3. Squeeze a small amount of cream on to your finger, and, using a swiping action, apply it on the affected area.
4. On contact with the saliva the cream will activate and create a protective layer over the affected area.
5. Re-apply the cream as necessary to maintain a constant protective layer over the affected area until cured.
6. As a rule of thumb, it generally takes two to three days for the ulcer to heal.

Note: You may notice a very slight 'sting' or 'burning sensation' when the cream is applied. Please note that this is completely normal and part of the healing action of the cream.

Precautions for use

- Avoid rubbing your tongue against the area in your mouth where the cream is applied.
- To avoid the tube clogging, always replace the cap tightly after use.
- Avoid bringing the tube nozzle into contact with humidity as this could clog the tube opening. If this does happen you can safely unplug it using a needle.
- Do not use the product if allergic to any of the ingredients mentioned on this instruction sheet and product box.
- Avoid ingesting the product in large quantities.
- If the product comes in contact with the eyes, rinse immediately with plenty of water. Seek medical attention if any irritation or symptoms persist.
- An overdose may cause side effects. Should any symptoms persist, seek medical attention.
- Keep out of reach and sight of children.
- Not recommended for children under the age of 12.

Pregnancy & breastfeeding

- If you are pregnant or breastfeeding, please consult your doctor before using the product.

Storage & Expiry

- Store cool and dry. (Avoid humidity)
- Do not use the product after the expiry date mentioned on the fold of the tube and on the box.

Caution: Do not ingest. Avoid contact with eyes. If in eyes rinse with water. If any irritation persists, seek medical attention. Keep out of reach of children. Store cool and dry.